<p>Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux. This product is conform with the directive 93/42/EEC for medical products. Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur. Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos. Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medicali. Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte. Соответствует требованиям директивы 93/42 ЕС о медицинских изделиях.</p>	
<p>Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :</p>	<p>381532 - 381533 381535 - 381536 381537 - 381538 381539 - 381933 381934 - 381935 381936 - 381938 381940</p>
<p>Fonction Function Functie Función Funzione Função Funktion Назначение</p>	<p>Permet à l'utilisateur de prendre appui en toute sécurité. Allows the user to lean safely. Zo kan de gebruiker in alle veiligheid ondersteund worden. Permite al usuario apoyarse con seguridad. Permette di prendere appoggio con sicurezza. Permite ao utilizador apoiar-se em toda a segurança. Zum sicheren Abstützen des Benutzers. Служит для пользователя надежной опорой.</p>
<p>Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание</p>	<div style="display: flex; align-items: center;"> <div style="margin-left: 20px;"> <p>150 kg</p> </div> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>120 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>90 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>80 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,600 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,920 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>1,050 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>1,8 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>2,500 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,500 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,580 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,670 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,750 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>0,990 kg</p> </div> <div style="display: flex; align-items: center; margin-top: 20px;"> <p>1,200 kg</p> </div>

<p>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции</p>	A38153201
<p>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания</p>	14/05/2014
<p>Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции</p>	04
<p>Date / Date / Datum / Fecha / Data / Data / Datum / Дата</p>	08/12/2015

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Do not install this equipment before reading the following instructions:
Fitting instructions. Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The 6 / 9 fixings points have to be necessarily respected. **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. The disinfection has to be made according to the recommendations of use of the various products of disinfection. After use of a cleaning product, make a rinsing in the clear water. **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user; use it for assistance **ONLY. DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar. At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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