Référence G / Reference G / Referentie G / Código G / Codice G / Referência G / Referenz G :	134707 134747																								
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	047626 047627																								
Fonction Function Functie Función Funzione Função Funktion Назначение	Permet de prendre une douche en position assise. Allows to take a shower in a seated position. Zo kan u zittend een douche nemen. Permite tomar una ducha en posición sentada. Permette di prendere una doccia in posizione seduta. Permite tomar um duche sentado. Ermöglicht es, im Sitzen zu duschen. Позволяет принимать душ в положении сидя.																								
Descriptif Description Beschrijving Descripción Descrição Beschreibung Описание	<table border="1"> <tr> <td></td> <td>047626</td> <td>150 kg</td> </tr> <tr> <td></td> <td>047627</td> <td>150 kg</td> </tr> <tr> <td></td> <td>134707</td> <td>150 kg</td> </tr> <tr> <td></td> <td>134747</td> <td>150 kg</td> </tr> <tr> <td></td> <td>047626</td> <td>2,540 kg</td> </tr> <tr> <td></td> <td>047627</td> <td>3,000 kg</td> </tr> <tr> <td></td> <td>134707</td> <td>2,600 kg</td> </tr> <tr> <td></td> <td>134747</td> <td>2,700 kg</td> </tr> </table>		047626	150 kg		047627	150 kg		134707	150 kg		134747	150 kg		047626	2,540 kg		047627	3,000 kg		134707	2,600 kg		134747	2,700 kg
	047626	150 kg																							
	047627	150 kg																							
	134707	150 kg																							
	134747	150 kg																							
	047626	2,540 kg																							
	047627	3,000 kg																							
	134707	2,600 kg																							
	134747	2,700 kg																							
Numéro notice / Instruction sheet no / Fichenummer / Número instrucciones / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	S00001353																								
Date de création / Creation's date / Aanmaakdatum Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	13/05/2015																								
Révision n° / Revision no. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / No редакции	01																								
Date / Date / Datum / Fecha / Data / Datum / Дата	02/05/2016																								

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Do not install this equipment before reading the following instructions:

Fitting instructions. Before drilling into the wall, physically hold the product (in the lowered position) against the wall so that the stand is PERPENDICULAR to the floor; then mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The

4 fixings points have to be necessarily respected. **Care and maintenance.** IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **Safety warnings.** The shower seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The shower seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. When the seat is in a raised position, DO NOT USE it as a grab bar. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this shower seat. At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS
Distributeurs : GODONNIER SAS & PELLET ASC