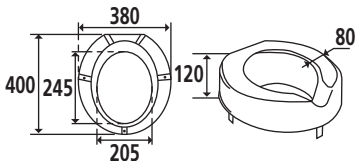
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	047570
Référence G / Reference G / Referentie G / Referencia G / Codice G / Referência G / Referenz G / Код G :	004757
Fonction Function Funcie Función Funzione Função Funktion Назначение	Permet de rehausser la cuvette des toilettes. Allows to raise the WC bowl Om de toiletpot te verhogen. Permite el realce del WC Permette di rialzare el WC Permite subir a sanita. Zur Erhöhung der Toilettenschüssel Предназначена для увеличения высоты унитаза.
Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание	  1,238 kg
Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	500001499
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	20/12/2002
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции	04
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	07/07/2015

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Do not install this equipment before reading the following instructions:

Fitting instructions. Put physically the product on the WC bowl in order to adjust support brackets. The fixings points have to be necessarily respected.

Care and maintenance. IMPORTANT: In order to guarantee the lasting quality of this product and its safety for the user; we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water.

Safety warnings. The raised toilet seat adds security to the user if it is fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. The toilet seat is designed to be used in seated position. To guarantee the user's safety, it is not advised to rise up on the seat. To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this toilet seat.

At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS
Distributeurs : GODONNIER SAS et PELLET ASC