Poids maximum utilisateur / Max. user weight / Maximengewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.

Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Référence P / Reference P / Referentie P / Referencia P / Codice P / Referência P / Referenz P / Код P :	046130	046140
	046150	046160
	046180	046200
	046220	046270

<b>Fonction</b>	Permet à l'utilisateur de prendre appui en toute sécurité.
<b>Function</b>	Allows the user to lean safely.
<b>Functie</b>	Zo kan de gebruiker in alle veiligheid ondersteund worden.
<b>Função</b>	Permite al usuário apoiar-se com segurança.
<b>Funzione</b>	Permette di prendere appoggio con sicurezza.
<b>Função</b>	Permite ao utilizador apoiar-se em toda a segurança.
<b>Funktion</b>	Zum sicheren Abstützen des Benutzers.
<b>Назначение</b>	Служит для пользователя надежной опорой.

<b>Descriptif</b> <b>Description</b> <b>Beschrijving</b> <b>Descripción</b> <b>Descrizione</b> <b>Descrição</b> <b>Beschreibung</b> <b>Описание</b>			046130	150 kg
			046140	150 kg
		046150	150 kg	
		046160	150 kg	
		046180	150 kg	
		046200	120 kg	
		046220	150 kg	
		046270	150 kg	
		046130	0,581 kg	
		046140	0,645 kg	
		046150	0,753 kg	
		046160	0,817 kg	
		046180	0,993 kg	
		046200	1,164 kg	
		046220	1,090 kg	
		046270	1,250 kg	

<b>Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции</b>	A55182801
<b>Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания</b>	27/01/2014
<b>Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакции</b>	03
<b>Date / Date / Datum / Fecha / Data / Data / Datum / Дата</b>	21/07/2015

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**GB** **Do not install this equipment before reading the following instructions:**  
**Fitting instructions.** Advised height 80 cm from the ground. Straight grab bars can be installed obliquely (135°) or horizontally. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. Plugs have to be selected according to the type of wall. The fixings points have to be necessarily respected. **Care and maintenance.** **IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. **DO NOT** use grab bar as a step to reach area, do not sit or hang on grab bar; use it for assistance **ONLY.** **The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar.** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS