kg Poids maximum utilisateur / Max. user weight / Maximumgewicht gebruiker / Peso máximo utilizador / Peso max. Utente/Peso máximo do utilizador / Maximales Benutzergewicht / Макс. вес пользователя.  
 Poids du produit / Product's weight / Productgewicht / Peso del producto / Peso del prodotto / Peso do produto / Gewicht des Produktes / Вес изделия

Ce produit est conforme avec la directive 93/42/EEC des dispositifs médicaux.  
 This product is conform with the directive 93/42/EEC for medical products.  
 Dit product beantwoordt aan Richtlijn 93/42/EEC over medische apparatuur.  
 Este producto es conforme con la Directiva 93/42/EEC relativa a los productos médicos.  
 Questo prodotto è in conformità con la Direttiva 93/42/EEC relativa ai dispositivi medici.  
 Este produto está conforme à Directiva 93/42/EEC relativa aos dispositivos médicos  
 Dieses Erzeugnis entspricht der Richtlinie 93/42/EEC über Medizinprodukte.  
 Соответствует требованиям директивы 93/42 ЕС о медицинских изделиях.

Référence G / Reference G / Referentie G / Referencia G / Codice G / Referència G / Referenz G / Код G :	488601
Référence P / Reference P / Referentie P / Referencia P / Codice P / Referència P / Referenz P / Код P :	048860 048865 048870 048875

Fonction	Permet à l'utilisateur de prendre appui en toute sécurité.
Function	Allows the user to lean safely.
Funcție	Zo kan de gebruiker in alle veiligheid ondersteund worden.
Función	Permite al usuario apoyarse con seguridad.
Funzione	Permette di prendere appoggio con sicurezza.
Função	Permite ao utilizador apoiar-se em toda a segurança.
Funktion	Zum sicheren Abstützen des Benutzers.
Назначение	Служит для пользователя надежной опорой.

Descriptif Description Beschrijving Descripción Descrizione Descrição Beschreibung Описание			150 kg
			1,977 kg

Numéro notice / Instruction sheet No. / Fichenummer / Número del manual / Numero manuale / Número do manual / Gebrauchsanweisungsnummer / Номер инструкции	A48860825
Date de création / Creation date / Aanmaakdatum / Fecha de creación / Data di creazione / Data de criação / Erstellungsdatum / Дата создания	08/04/2015
Révision n° / Revision No. / Revisie nr. / Revisión n° / Revisione n° / Revisão n° / Revisionsnr. / № редакци	01
Date / Date / Datum / Fecha / Data / Data / Datum / Дата	26/01/2016

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**GB Do not install this equipment before reading the following instructions:**  
**⚠️ Fitting instructions.** Advised height for the upper bar: 80cm in toilets and showers, or at the height of the washbasin. Before drilling into the wall, physically hold the product against the wall and mark the position of the holes to be drilled. On walls unable to support sufficient weight, there are 3 possible solutions: use of the vertical support post, the stand, or a reinforcement plate fitted behind the wall. Screws and plugs should be selected according to the type of wall. The 4 fixing points have to be necessarily respected. **⚠️ Care and maintenance. IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases, esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **⚠️ Safety warnings.** Grab bars add security to the user if they are fitted according to instructions. After fitting and before using, make sure that the fixing screws are tightened securely. Grab bars are not designed to support the total weight of the user; use it for assistance ONLY. DO NOT use grab bar as a step to reach area, do not sit or hang on grab bar. **Use it for assistance ONLY.** To avoid any pinching risks, do not put fingers in the mobile parts. The product does not contain any hazardous substances. The manufacturer shall have no liability for any injury caused by improper installation or use of this grab bar. **⚠️** At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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**GB** **Do not install this equipment before reading the following instructions:**

**Fitting instructions.** This armrest can be adapted on all our grab bars Ø 30 mm, minimum length 400 mm. For the fitting, see drawings below. **Care and maintenance.** **IMPORTANT:** In order to guarantee the lasting quality of this product and its safety for the user, we recommend that when cleaning it you should use mild soap solutions. **CAUTION: DO NOT USE** cleaning products made of: etones, hydrocarbons, acids, bases,

esters and ethers. After use of a cleaning product, make a rinsing in the clear water. **Safety warnings.** In order to be effective, the armrest must be installed according to the instructions. This armrest has been studied to be mounted once, do not dismount. The manufacturer shall have no liability for any injury caused by improper installation or use of this armrest. The product does not contain any hazardous substances. At the end of its life cycle, the user must have the product recycled by a certified facility in the country of use.

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Fabricant : GODONNIER SAS
Distributeurs : GODONNIER SAS et PELLET ASC